




# Face shield

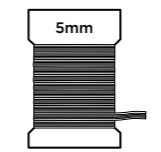


Elastic band (5mm in width)

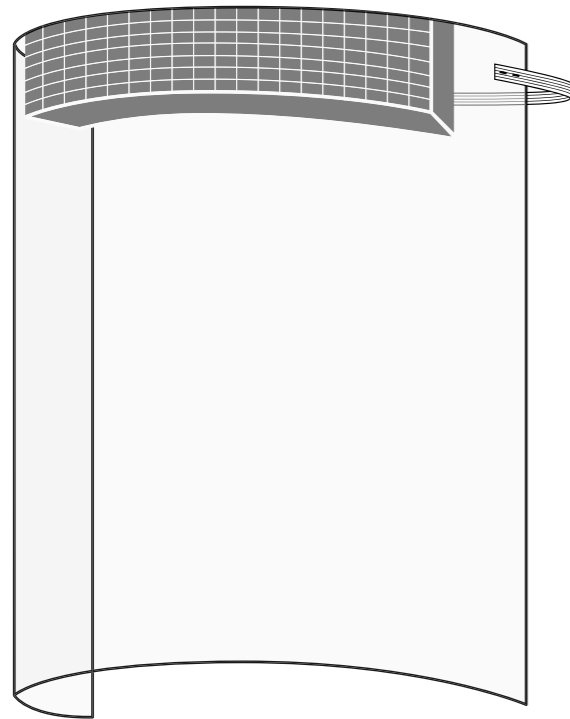


A4 overhead projector film

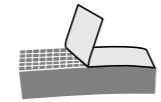

Foam strip (30x30x200)

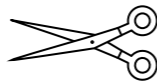


Scissors

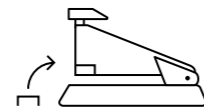


Staples (standard 26/6)



Tape to fix the templates on the table

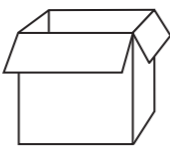

Boxes to pack

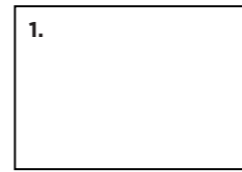

A3 templates x 5 (printed in full size 100%)

## Before starting:





Only those who feel completely healthy can participate



## It is important to:



- wear a face mask
- clean and disinfect tables, staplers and scissors
- wash your hands with soap and use hand disinfectant

1.

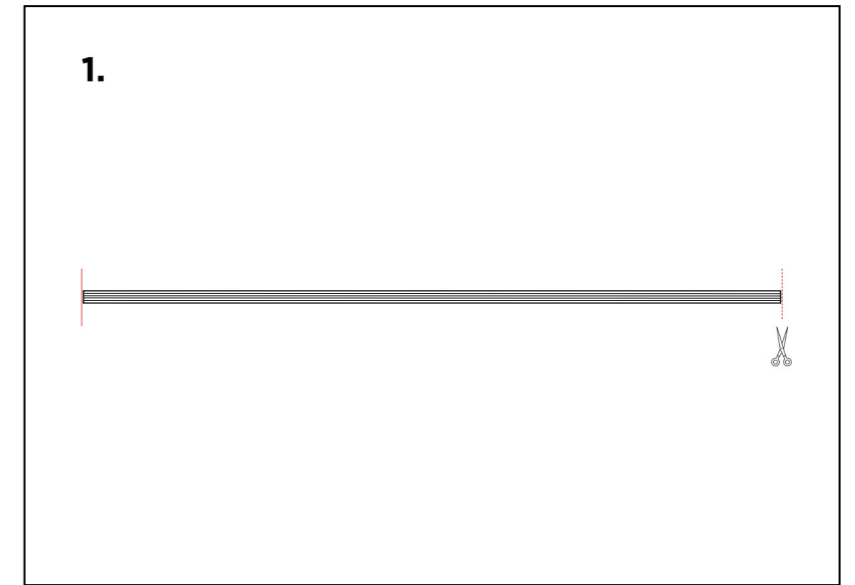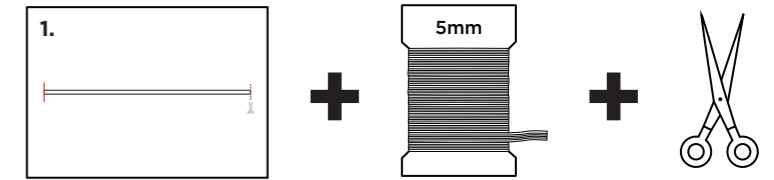


Cut the elastic band (36 cm)

2.

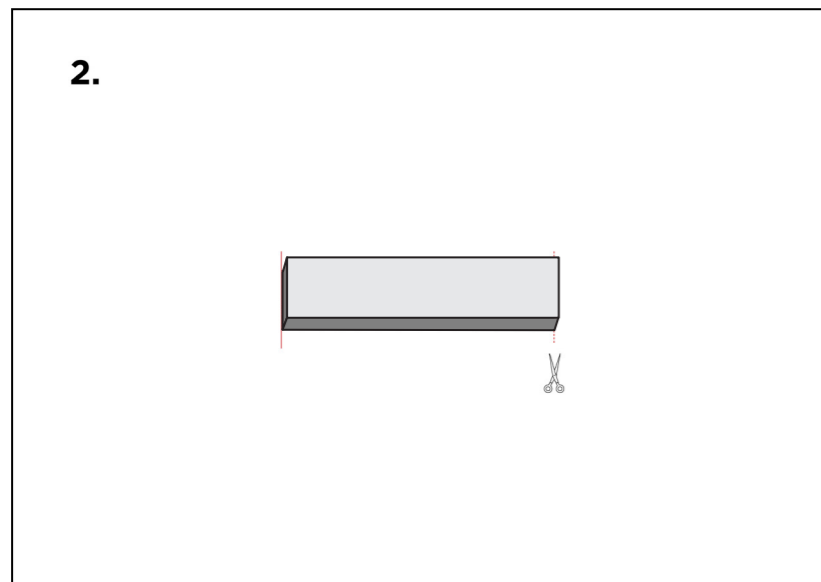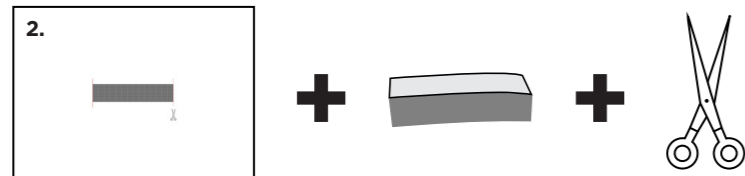



Cut the foam strip (14 cm)

3.

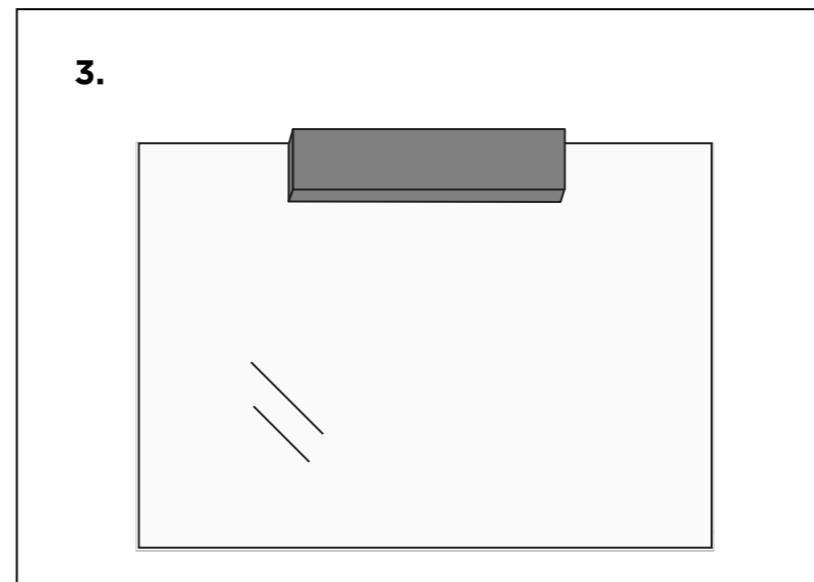

Stick the foam onto A4 overhead film

4.

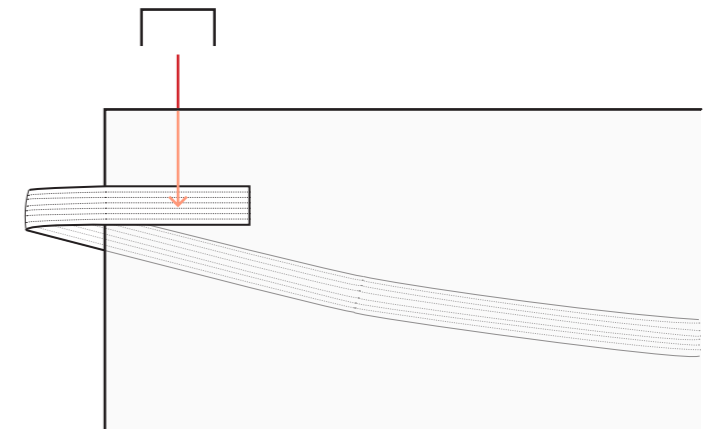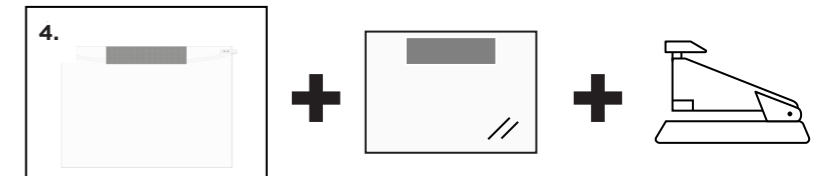


Staple the elastic band onto the A4 overhead film

5.

Check the security of the face shields by looking at the position of the staples and pulling the elastic band.


1.

Use this template to cut the elastic band in the right size.

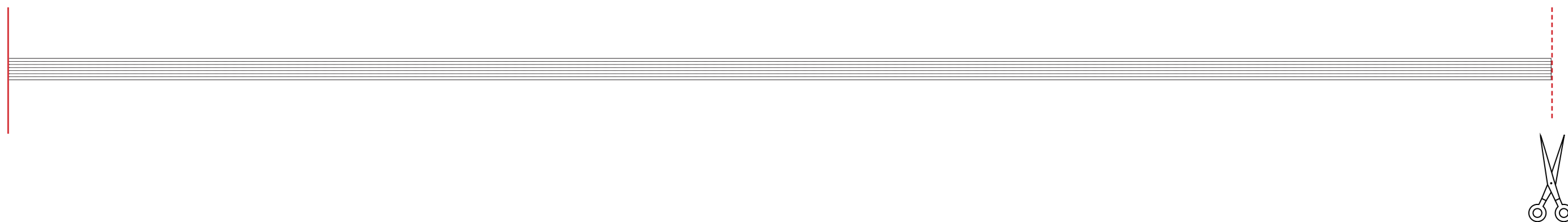


# 2.

Use this template to cut the foam strip in the right size.

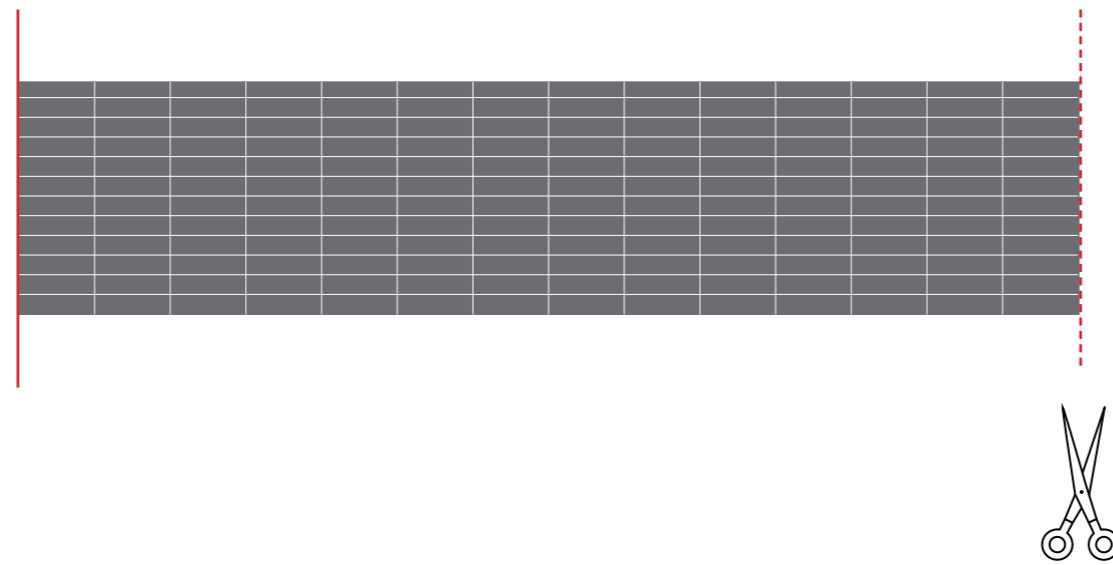


**3.**

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Use this template to stick the foam in the middle of the overhead projector film.



4.

Connect the elastic band onto the outside of the film, by stapling from the inside.

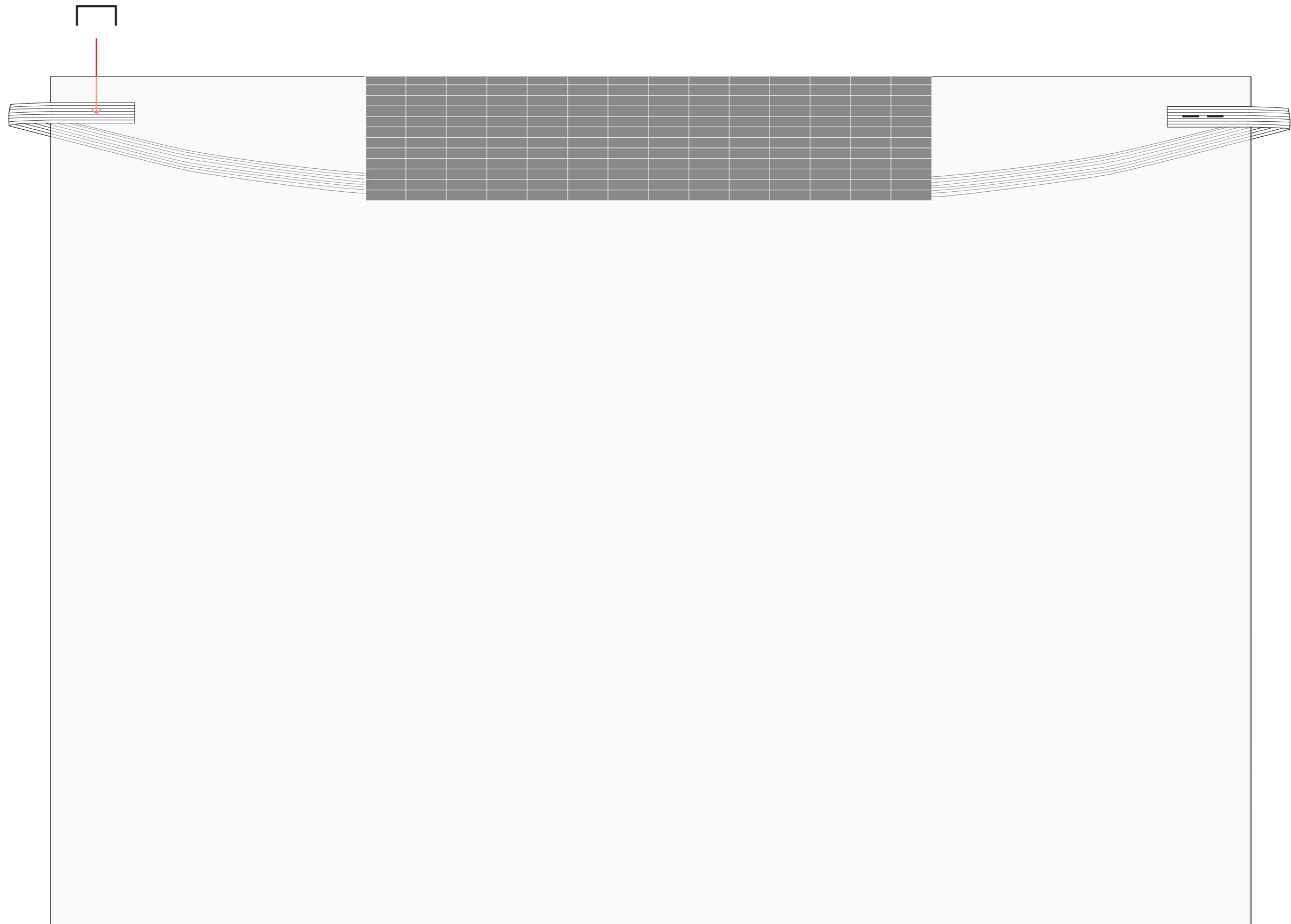

# 5.

Before you pack, check the security of the face shields by looking at the position of the staples and pulling the elastic band.

